


# Slides for Pediatric Model

## Preschool Children

The "E" points to a Bird, a Rabbit, a Girl or a Boy.




**Test No. 1, SMPS-B2 —Binocularity**  
Tests performance of both eyes together.

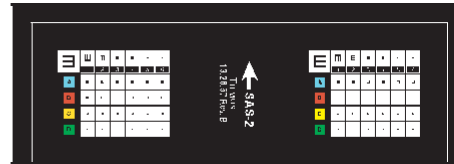


**Test No. 5, SAS-2 —Acuity Right and Left Eye**

Conventional Tumbling "E" symbol. Child states which way the "E" points in each square. Acuity levels of 20/20, 20/30, 20/40 and 20/50.

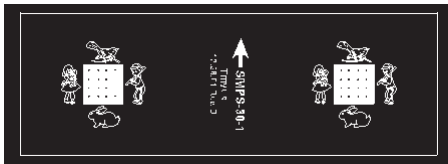


**Test No. 2, SMPS-30-1 —Acuity Right and Left Eye, 20/30**

The "E" points in a different direction for each eye.

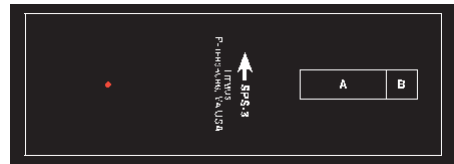


**Test No. 6, SPS-3 —Heterophorias**

Muscle imbalance — vertical and lateral — is tested at predetermined pass/fail levels. This test is administered first at far, then at near.

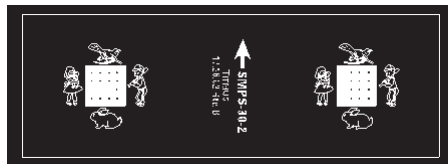


**Test No. 3, SMPS-30-2 —Acuity Right and Left Eye, 20/30**

The "E" points in a different direction for each eye.

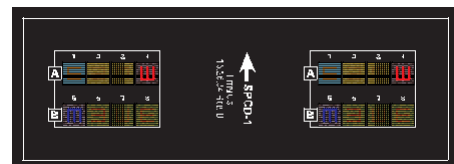

**Test No. 7, SPCD-1 — Color Perception**

Titmus Pseudo-Isochromatic Plates reliably detect gross color anomalies, but do not classify the type of deficiency.

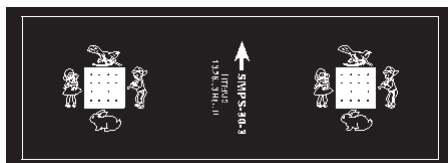

**Test No. 4, SMPS-30-3 —Acuity Right and Left Eye, 20/30**

For each eye, all the "E" s point to a different object.  
Also available 20/40 acuity.



**Test No. 8, SA-BRL-4 —Visual Acuity**

Results calibrated at 20/20, 20/30, 20/40, 20/50, 20/70, 20/100 and 20/200. Tests can be administered binocularly or with either eye occluded.

Improvements have been made to test slides to enhance viewing performance. **Patent Pending.**